

# FLEX PIL-MA

HYDROPHILIC PRELOADED



3 CLICKS FOR EASY,  
SAFE AND PREDICTABLE  
INJECTION


**MEDI**  **NTUR**

Vision of Expertise



# FLEX PIL-MA

Single use injection system  
for Q-Flex preloaded  
monofocal and diffractive  
progressive hydrophilic IOLs

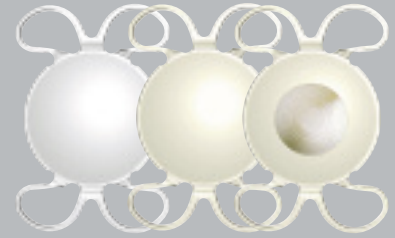


### 1<sup>st</sup> CLICK LOADING

Insert the injector to the  
wet container




### 2<sup>nd</sup> CLICK CLOSING

Fold up the cartridge  
nozzle

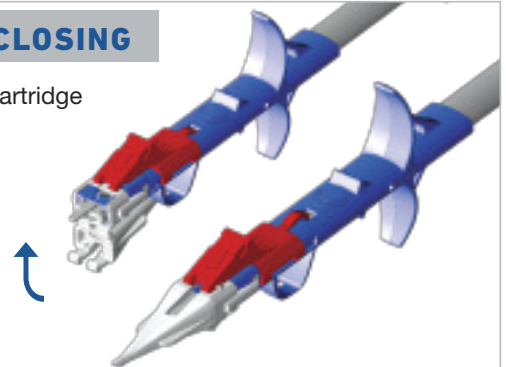


### 3<sup>rd</sup> CLICK READY

Push the red  
stopper forward  
to the end  
position and  
fill visco

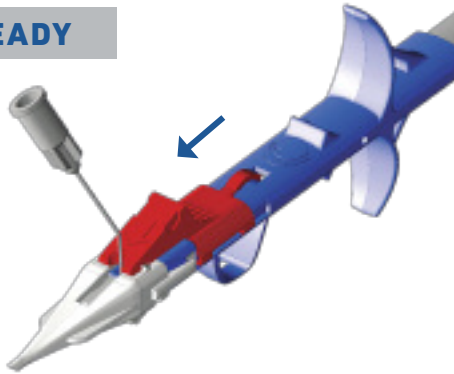

### INJECTION

Push the plunger  
slowly to the  
end position

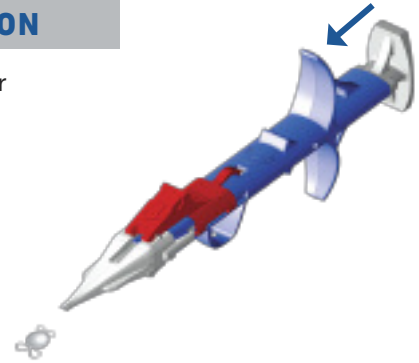

#### Lens types

Single-piece aspheric IOLs,  
clear and yellow  
• monofocal  
• diffractive progressive

(Patent pending)

**Optic diameter** 6.0 mm

#### Overall diameters

0.0 D → +15.0 D · 11.0 mm  
+15.5 D → +22.0 D · 10.7 mm  
+22.5 D → +30.0 D · 10.5 mm

**Angulation** 0°

**Material** Copolymer of hydrophilic  
and hydrophobic monomers

**Incision size** 2.2 mm

**PCO protection** 360° Special Square Edge

**A-Constant** 118.9 (SRK/T)



#### 640P/640PY

25% water content  
with UV Absorber /  
+ blue light filter (390–470 nm)

Powers available

0.0 D → +30.0 D · (0.5 D steps)



**640PMY** 25% water content  
with UV Absorber /  
+ blue light filter (390–470 nm)

Powers available

0.0 D → +30.0 D · (0.5 D steps)

Addition 3.0 D; +3.5 D

LA-IMCQPLMAENG 02 201605