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EN

DESCRIPTION

VisuECTOINE® is a humectant and lubricating sterile ophthalmic solution based on ectoine, sodium hyaluronate and sodium carboxymethylcellulose in the form of a hydrogel. VisuECTOINE® is a sterile eye drop, pH-buffered, phosphates free and without preservatives.

COMPOSITION

VisuECTOINE® is formulated with ectoine, sodium carboxymethylcellulose, high molecular weight sodium hyaluronate, pharmacopeia quality salts and purified water:

Content	%(w/w)
Ectoína	2
Sodium hyaluronate	0.1
Sodium carboxymethylcellulose	0.15
Disodium tetraborate dehydrated	0.1
Boric acid	0.8
Purified water	c.s.p. 100

PACKAGING

VisuECTOINE® is supplied in single-use containers made of low density polyethylene containing 0.4 ml of the product. Each box contains 30 single-use containers.

INTENDED USE

VisuECTOINE® is an ophthalmic solution indicated to alleviate the ocular dryness and irritation produced by environmental factors (wind, dust, smoke, air conditioning, dry heat), prolonged use of screens or contact lenses and/or some pathologies that present the above mentioned symptoms.

PROPERTIES

VisuECTOINE® alleviates eye dryness and dry eye sensations. VisuECTOINE® alleviates eye irritation. VisuECTOINE® protects against external allergens alleviating allergy symptoms.

VisuECTOINE® is compatible with contact lenses.

DOSAGE AND ADMINISTRATION

VisuECTOINE® is for ophthalmic use (applied to the surface of the eye).

Before opening a new VisuECTOINE® container, ensure that the seal of the plug is intact by checking the integrity of the safety ring and the plug.

The solution must be used immediately after opening. In order to avoid tip or product contamination do not touch the tip of the vial on the eye or other surfaces.

If you find you are having difficulties in handing the vials it is recommended you ask for assistance in the correct administration of the product.

The recommended dose is one or two drops in each eye, as many times as required or according to the instructions received by the specialist.

DIRECTIONS FOR USE

1. Wash and dry your hands before using the product.
2. Remove a single-use vial from the strip.
3. Verify that the vial is not chipped or cracked.
4. Turn over the plug to open the vial.
5. Remove the plug.
6. Tilt your head back and look up.
7. Gently pull the lower eyelid to form a small pocket between the eyelid and the eye.
8. Hold the container with the other hand between the thumb and the middle finger and gently press to drop one or two drops of VisuECTOINE® into the eyelid pocket.
9. Blink several times to allow the drop to distribute over the entire surface of the eye.
10. If the drop did not enter the eye, apply again.
11. Carefully wipe any residue that may be around the eyelid with a gauze.
12. Repeat the procedure with the other eye.
13. Discard the single-use vial (once opened do not re-use the vial for the next doses).

CONTRAINDICATIONS

Do not administer VisuECTOINE® if you have hypersensitivity to hyaluronic acid or other components of the product.

WARNINGS AND PRECAUTIONS

- Do not use if you are allergic to any ingredient.
- Do not touch your eye, or any surface with the tip of the product vial.
- Do not share the product with other people.
- The product may cause temporary blurring immediately after using the drops. Wait for clear vision before carrying out dangerous activities.
- Do not use the product after the expiration date indicated on the box or on the packaging.
- Keep out of the sight and reach of children.
- Use with caution in pediatric population (product not evaluated in children).
- For external use only, do not ingest.
- Consult your doctor, pharmacist or optician before using the product during pregnancy or breast-feeding.

INTERACTIONS

Do not use with other products containing quaternary ammonium salts such as benzalkonium chloride because sodium hyaluronate may precipitate in its presence.

After using VisuECTOINE®, wait at least 15 minutes to apply any other product or medicine to the eye.

VisuECTOINE® is compatible with contact lenses. After putting on the contact lenses and before applying the drops, it is recommended to wait about 30 minutes to avoid incompatibility reactions between the products used for the maintenance of contact lenses and VisuECTOINE®.

ADVERSE EVENTS

VisuECTOINE® is well tolerated.

In some cases it may cause temporary eye irritation, blurred vision, foreign body sensation or eye hyperemia. Contact your doctor or pharmacist if you notice any side effects related to using VisuECTOINE®. If these effects persist or worsen, consult your doctor or pharmacist.

STORAGE

Store between 5°C and 25°C. Do not freeze. Store the vials in the original packaging and keep away from light.

EXPIRY DATE

Do not use after the date marked on the package.

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